

truCOLLECT[®] Whole Blood Collection Solution

Instructions for Use

Scan the QR code
to learn more
about truCOLLECT
bit.ly/45EXvuN



Preparation Before Collection


- Drink 16 oz (475 mL) of water, 45 minutes prior to collection
- Wash your hands with soap and warm water until your hands are warm
- Select a clean, flat surface for your collection device
- Standing during collection is preferred, but if you want to sit, your hand must be below your heart

Collection Kit Contents


- Instructions for Use
- Convenience Kit Bag
 - Alcohol Wipe
 - Gauze
 - Adhesive Bandage (2)
 - Lancet (2)
- Desiccant Cap Pouch
 - Orange Tool & Desiccant Cap on Clear Tube
- Collection Device Pouch
 - Collection Device
 - Blue Cap

Before beginning, please read all directions before starting. Remove items from Convenience Kit Bag and Collection Device Pouch, place in their designated spot to the right.


1. Place **Collection Device** horizontally on a flat surface. **Capillary** must be parallel to flat surface.

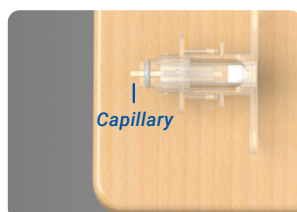


2. Select a **puncture site** on ring or middle finger and disinfect with **Alcohol Wipe**.

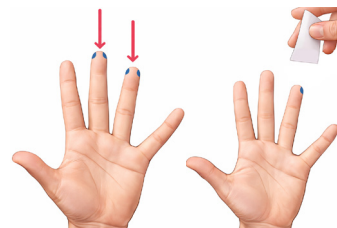


3. Prepare **Lancet** by twisting off tab (**discard tab**).

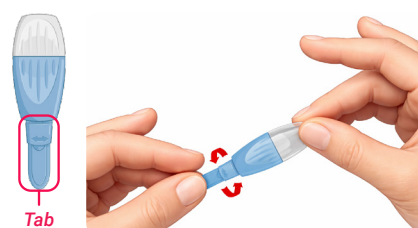


4. Place open end of **Lancet** against **puncture site**; press firmly until you hear a **CLICK**. Wipe 1st drop of blood with **Gauze**.

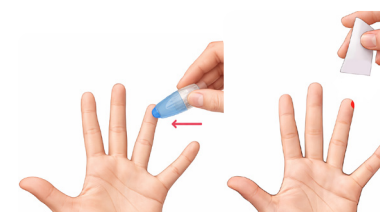


5. Starting with the 2nd drop of blood, collect in **Capillary** and fill to the end (**marked by 2 Fins on Collection Device**). Leave horizontally on flat surface.

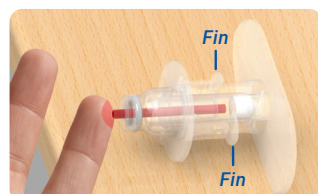


6. Use **Gauze** to wipe remaining blood on puncture site and apply **Bandage**. If unable to fill **Capillary**, repeat **Steps 2-4** on a new site. Confirm **Capillary** is filled to end.

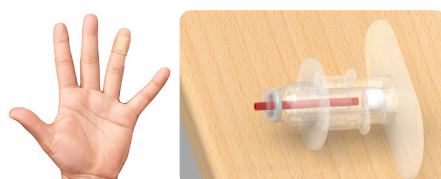

7. Place **Collection Device** on flat surface with end of **Capillary** pointing upward.

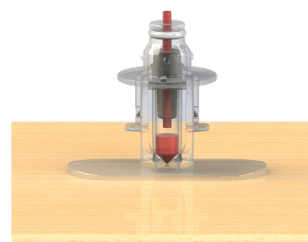


8. Securely hold base of **Collection Device**. Attach **Blue Cap** to the top and slowly turn clockwise, twisting until you hear a **CLICK**. After the click, pull the **Blue Cap** upward and discard.

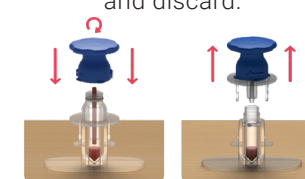


9. Open **Desiccant Cap Pouch**. Turn **Orange Tool** counter-clockwise to remove **Desiccant Cap** from **Clear Tube**. If **Orange Tool** detaches, press back on. Discard **Clear Tube**.




10. Insert **Orange Tool** & **Desiccant Cap** into open **Blood Tube** in the **Stand**. Turn **Orange Tool** clockwise until **Blood Tube** is closed tightly.

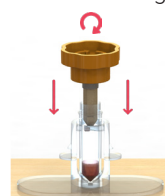

11. Allow closed **Blood Tube** to remain in upright position for **60 minutes**.

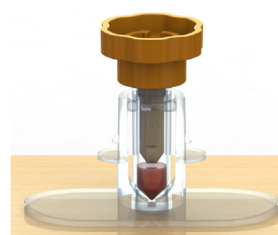

12. After **60 minutes**, pull **Blood Tube** up and out of the **Stand**. Discard **Stand**, and ship **Blood Tube***.

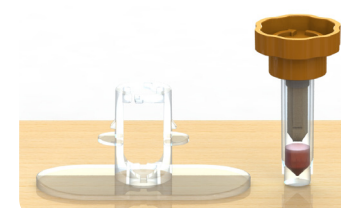

Adhesive
Bandage
(1 of 2)

Adhesive
Bandage
(2 of 2)

Collection
Device
(Discard after
final use)

Blue Cap
(Discard after
final use)

Lancet
(1 of 2)
(Discard Tab
after use)

Gauze
(Open before collection)

Alcohol Wipe
(Open before collection)

Lancet
(2 of 2)
(Discard Tab
after use)



Read Directions Before Use



Single Use Only

*Ship in accordance with supplied conditions. Patent protected.
Information subject to change without notice.



truCOLLECT[®]

Whole Blood Collection Solution

Instructions for Use

Scan the QR code
to learn more
about truCOLLECT
bit.ly/45EXvuN



Read Directions Before Use



Single Use Only

*Ship in accordance with supplied conditions. Patent protected.
Information subject to change without notice.



US: +1 781.932.3959
customerservice@covaris.com

EU: +44 (0)845 872 0100
emeacustomerservice@covaris.com

APAC: +86 137 6427 6714
APACcustomerservice@covaris.com

Applications

applicationsupport@covaris.com

Service and Support

techsupport@covaris.com

www.covaris.com | 2026© Covaris, LLC
010562_RevB_Feb2026

Scan the QR code
to learn more
about truCOLLECT
bit.ly/45EXvuN

